
E A R W E A R

EW 1104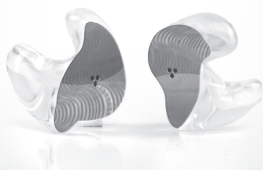

User information

EN

Dear customer,

Please read this user manual carefully before using your hearing protector for the first time and store it in your case. It provides valuable information on your product. Protect your hearing for noise-induced hearing loss cannot be cured. Noise is unwanted sound which not only annoys us but which can cause many illnesses. Noise-induced hearing loss is much more than just an occupational risk but more often the result of ear-damaging leisure activities. Unfortunately, the use of earplugs is very often thoughtlessly abandoned. Most frequently, the reason for this is lack of comfort. Earwear EW 1104 offers safe hearing protection and excellent wear comfort!

General information:

- ▶ Earwear EW 1104 is a reusable individually adjusted earplug made from tear-proof and hypo-allergenic material.
- ▶ You will find a colourful marking on the right otoplastics to distinguish between the sides.
- ▶ Caution: Your earplug can only achieve its intended protective function if inserted properly and worn rationally in the noise area.
- ▶ Is tested according to EN 352-2:2002 and CE certified acc. to the PPE Directive EU 2016/425 by PZT GmbH, An der Junkerei 48F, 26389 Wilhelmshaven, #1974
- ▶ Reference to the declaration of conformity: <https://www.earwear.me/service/produktdaten>
- ▶ Reference to the harmonised standards: https://eur-lex.europa.eu/legal-content/DE/TXT/?uri=uriserv:OJ.C_.2018.209.01.0017.01.DEU
- ▶ Reference to this Regulation: <https://www.ris.bka.gv.at/eli/bgl/II/2007/196/20070807>

Insertion:

- ▶ Take the earplug between your thumb and index finger and insert it into your ear canal, using gentle pressure.
- ▶ Using a gentle turning motion, insert the earplug deeper into your ear canal.

Removal:

Grab the earplug with your thumb and index finger and pull it out of your ear.

Impairment of the earplug's function:

- ▶ Regularly check your earplug for damages and whether maintenance may be required.
- ▶ Have your ear protection product checked by an expert every two years.
- ▶ We generally recommend replacing your earplug after 4 years at most.

Cleaning and care:

- ▶ Thoroughly clean your earplug with lukewarm water after every use and with mild soapy water in case of strong contamination.
- ▶ Certain chemical substances can have adverse effects on this product. Additional information can be requested at the manufacturer.
- ▶ Your customer service associate will inform you about recommended care products!
- ▶ Please use the provided case to store your product.

Warning notes:

- ▶ In case of pressure marks and feelings of pressure, please check your earplug's seat.
- ▶ Should you show allergic reactions such as red or itchy skin etc. please stop using your earplug and clarify the reason with a specialist physician.
- ▶ Keep away from toddlers. Risk of asphyxiation when swallowed!
- ▶ Please note that your earplug may impede your perception of possible hazards and warning signs. Therefore, pay special attention to possible hazards and warning signs in your surroundings when using your earplug.
- ▶ Sudden or rapid removal of the earplug from your ear canal may damage your eardrum.
- ▶ In case of ear injuries the earplug must not be used any more and we recommend seeking medical advice!

Service and warranty:

In case of warranty claims please contact your customer service associate with your receipt. Warranty claims shall be governed by statutory provisions. The warranty excludes all damage attributable to improper use, and non-compliance with the care instructions.

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Neuroth International AG